URGENT MEDICAL DEVICE CORRECTION

GE Healthcare

3000 N. Grandview Blvd. - W440
Waukesha, WI 53188, USA

<Date of Letter Deployment>

GEHC Ref# 32047

To: Nurse Managers, Labor & Delivery/NICU
Bio-Medical Engineering Department Managers
Risk Management Directors

RE: Infant Warmer System (IWS) Hot "Heater Head" Screw could Fall onto the Bed

GE Healthcare has recently become aware of a potential safety issue related to loose screws in the "Heater Head" of certain Infant Warmer System (IWS) devices. **Please ensure that all potential users, as well as those servicing these units, in your facility are made aware of this safety notification and the recommended actions.**

**Safety
Issue**

Hot screws from the "Heater Head" of the IWS could fall onto the bed if the "Heater Head" assembly has been improperly serviced. This situation can be clinically hazardous because thermal injury to a patient could result. Two injuries have been reported as a result of this issue.

**Safety
Instructions**

The potential for this issue to occur is only present if the "Heater Head" assembly has been improperly serviced.

As a result, if "Heater Head" service has been performed in the past, check to ensure that the screws are tightened as soon as the unit becomes available.


If the "Heater Head" assembly has not been serviced, check to ensure that the screws are tightened as part of your next regularly scheduled annual Preventive Maintenance check.

The attached Service Manual Addendum provides instructions for checking and tightening the screws. During each annual Preventive Maintenance check, continue to ensure that the screws are tight.

**Affected
Product
Details**

Refer to Product Images below for images of Affected and Not affected products.

Affected model numbers: Model 2001 IWS (International), 3000 IWS, 3050 IWS, 3100 IWS, 3150 IWS, 3300 IWS, 3500 IWS, 4000 IWS, 4300 IWS, 4400 IWS, 5000 IWS.

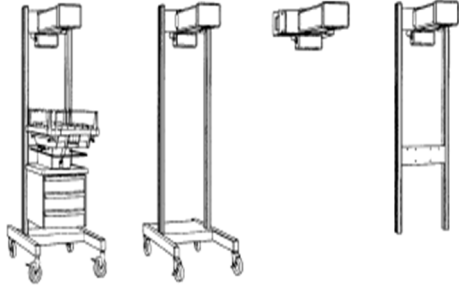Affected Products	Not Affected Products
	

**Product
Correction**

Attached to this letter, we provide instructions as part of a Service Manual Addendum on how to correct the issue. Please add this new Addendum to the Service Manual of your device(s) and train the affected users accordingly.

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
Please acknowledge that you have received this letter and that you understand that an action needs to be taken on your part to correct this issue by filling out and returning the attached "Customer Response" form.

**Contact
Information**


If you have any questions or concerns regarding this notification, please contact GE Healthcare Service at 1-800-437-1171 or your local Service Representative.

Please be assured that maintaining a high level of safety and quality is our highest priority. If you have any questions, please contact us immediately per the contact information above.

Sincerely,

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James W. Dennison
Vice President - Quality & Regulatory
GE Healthcare

Jeff Hersh, M.D.
Chief Medical Officer
GE Healthcare

GE Healthcare

**MEDICAL DEVICE CORRECTION CONFIRMATION
CUSTOMER RESPONSE REQUIRED**

GE REF: 32047

We request that you **PLEASE COMPLETE** and return this form to GE Healthcare within two (2) weeks.

Customer/Consignee Name: _____

Street Address: _____

City/State/ZIP/Country: _____

Email Address: _____

Phone Number: _____

It is important that we confirm our customers have received this correction notice. Please check one of the following and complete the requested information and send back via one of the methods below.

- ☐ We acknowledge receipt and understanding of the Medical Device Correction Notice and have alerted the appropriate personnel at our facility regarding the safety issue and instructions. We will perform the actions as requested in the attached Medical Device Correction Notice on all potentially affected systems.

List all Device/System Serial Number(s) known (attachment can be used): _____

- ☐ We acknowledge receipt and understanding of the Medical Device Correction Notice and no longer have a system affected by this Medical Device Correction Notice. (Please check appropriate disposition. If multiple systems or further information, attachment can be used.)

☐ Sold ☐ Returned ☐ Scrapped ☐ Other: _____

Device/System Serial Number(s): _____

New Owner, if known: _____

Contact Name: _____

Street Address: _____

City/State/Country: _____

Contact (i.e. Email, Phone): _____

Please provide the name of the individual with responsibility for risk and compliance.

Signature: _____

Printed Name: _____

Title: _____

Date (DD/MM/YYYY): _____

Please return this form using one of the following methods:

- Scan or take photo of completed form and email to MIC.Recall@ge.com**
Note: QR code can be used to email the form: click QR code, attach photo to email, click Send
- Take photo of completed form and send via SMS text to +1-410-972-8096**
Note: QR code can be used to text the form: click QR code, attach photo to text, click Send
- Fax completed form to Fax Number: +1-410-630-5938**

32047 – XXXX

QR (text)

QR (email)

Addendum

Infant Warmer System Heater Reflector Screws Maintenance Check

For future reference, add this addendum to your
Infant Warmer System Service Manual.

2098189-001 A

© 2016 General Electric Company

Language Disclaimer

(EN)

WARNING:

This service document is available in English only.

- If a customer's service provider requires a language other than English, it is the customer's responsibility to provide translation services.
- Do not attempt to service the equipment unless this service manual has been consulted and is understood.
- Failure to heed this warning may result in injury to the service provider, operator, or patient from electric shock, mechanical hazards, or other hazards.

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Maintenance Schedule

Perform this maintenance procedure **annually** to make sure the two heater reflector screws (front and back) are tight. This schedule recommends the minimum frequencies. Always follow hospital and local regulations.

WARNING: Failure to perform this maintenance per recommended frequency can result in patient harm or injury.

Required Tools

- One long Phillips screw driver
- One 5/16" wrench or nut driver
- One small step stool
- One 5/64" hex wrench
- One 3/8" nut driver

Heater Reflector Screws Maintenance Check Procedure

1. Turn off the Infant Warmer System and unplug its power cord from the outlet. Allow the heat tube to cool down before proceeding with the next steps.

WARNING: Disconnect the power to the warmer and allow the heat tube to cool to avoid the possibility of a burn.

2. Locate the warmer unit serial number on the back of controller.
3. For units that have serial numbers starting with the prefix HAK, AAN, HAJ, or HCA (see Figure 1):
 - a. Use a 5/64" hex wrench to remove the four mounting screws for the front cover plate, and remove the plate.
 - b. Use a 3/8" nut driver to remove the four mounting screws for the front heater housing cap, and remove the cap.
 - c. Proceed to step 5.

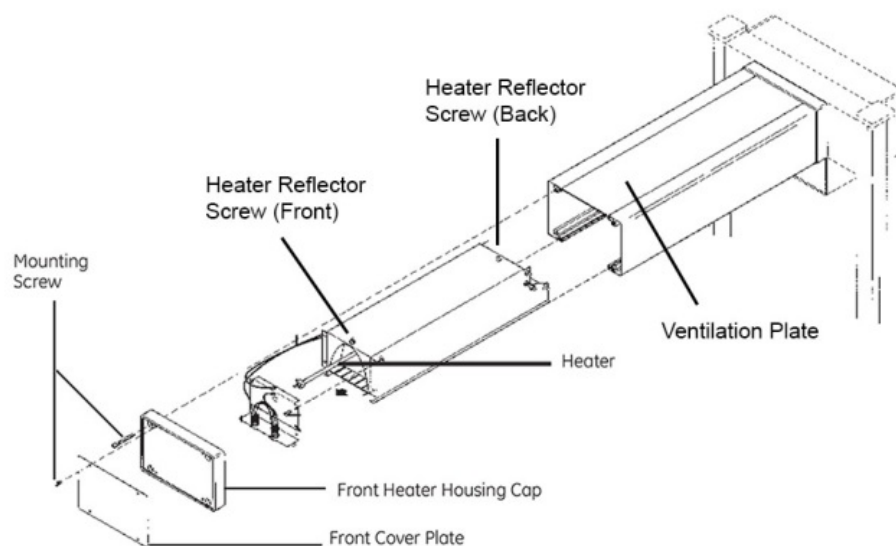
Figure 1: Heater Head Assembly – Units with HAK, AAN, HAJ, or HCA Serial Numbers

4. For units that have serial numbers starting with the prefix HCC (see Figure 2):
 - a. Use a Phillips screw driver to remove the screw and lock washer from the top of the cushioned heater head.
 - b. Use a Phillips screw driver to remove the screw and alarm light lens from the bottom of the heater head. Now the cushioned heater head can be pulled off the front of the unit.
 - c. Use a wrench to remove the four screws from the support plate end cap, and remove the plate.
 - d. Proceed to step 5.

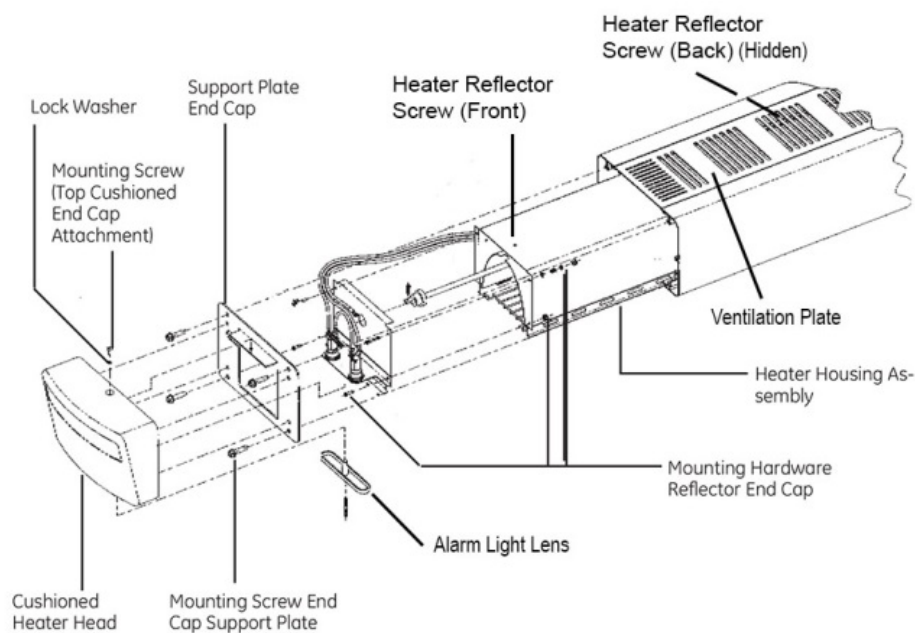
Figure 2: Heater Head Assembly – Units with HCC Serial Numbers

5. Pull out the ventilation plate on the top of the heater housing assembly (Figure 3).

Figure 3: Removing the Ventilation Plate

6. Locate the two reflector screw nuts (front and back) on the top of the heater housing assembly. (See Figure 4 and Figure 5).

Figure 4: Locating the Front Reflector Screw Nut

Figure 5: Locating the Back Reflector Screw Nut

7. Pass the tip of a long Phillips screw driver through the heater grille at an angle to engage the **front** reflector screw, and then use a wrench or nut driver to loosen the nut about one turn (Figure 6). Use care not to remove from screw. (You might need a small step stool to reach to the top of the heater assembly.)

Figure 6: Engaging the Front Reflector Screw

8. While keeping the screw engaged with the screw driver, use your fingers to tighten the nut until it is snug.
9. Use a 5/16" wrench or nut driver to tighten the nut about an additional one-third of a turn (Figure 7).

Figure 7: Tightening the Nut in the Front Reflector Screw

10. Pass the tip of a long Phillips screw driver through the heater grille at an angle to engage the **back** reflector screw, and then use a wrench or nut driver to loosen the nut about one turn (Figure 8). Use care not to remove from screw. (You might need a small step stool to reach to the top of the heater assembly.)

Figure 8: Engaging the Back Reflector Screw

11. While keeping the screw engaged with the screw driver, use your fingers to tighten the nut until it is snug.
12. Use a 5/16" wrench or nut driver to tighten the nut about an additional one-third of a turn.
13. Re-install the ventilation plate into the system. Push the side tabs down on the ventilation plate, if necessary, for proper installation (Figure 9).

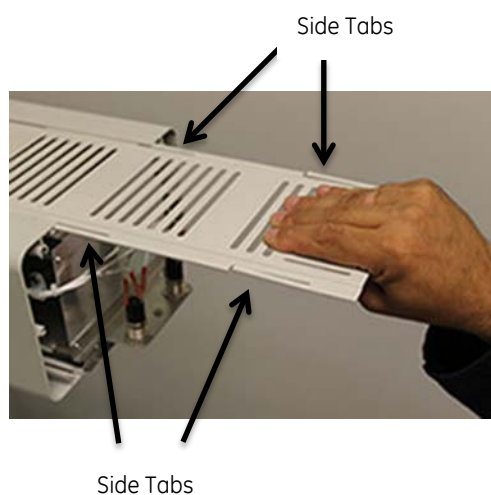
Figure 9: Re-installing the Ventilation Plate

14. Reverse step 3 or step 2 to re-install the heater head plate and cover.

NOTE: If the heater reflector screw or nut is lost, you can order the reflector screw by GE part number 0140-6124-114 and the nut by GE part number 6600-0382-400.

World Headquarters

GE Healthcare
9900 West Innovation Drive
Wauwatosa, WI 53226-4856
USA
Tel 1 800 345 2700

Europe, Middle East, Africa

GE Healthcare
P.O. Box 900
FIN-00031 GE
Finland
Tel +358 10 39411
Fax +358 9 146 3310

Latin America Representatives

GE Healthcare
3350 SW 148 Avenue
Suite 301
Miramar, Florida, 33027
USA
Tel + 1 954 744 5600

Brazil Only

GE Healthcare Clinical Systems Equipamentos Médicos Ltda
Av. Paulista, 37 - 13º andar
CEP: 01311-902 - Cerqueira César
São Paulo, SP - Brasil
Tel +55 11 3053 2500
Fax +55 11 3053 2573

EC Representative

GE Medical Systems SCS
283 Rue de la minière
78530 BUC - FRANCE

Germany

GE Medical Systems Information Technologies GmbH
Munzinger Str. 3-5
79111 Freiburg
Tel. 49 761 4543 570
Fax 49 761 4543 571
Service 0800 4343258

Asia Representative

GE Healthcare
Shanghai GE (China) Hi-tech Park
No1 Huatuo Road, Zhangjiang Hi-tech Park Pudong, Shanghai,
P.R.China 201203
上海GE中国科技园
地址：中国上海市浦东张江高科技园华佗路1号, 201203
Tel + (8621) 38777888
Fax + (8621) 38777402

Australia	1300 722 229
China	800 810 8188
India	1 800 425 7255
Korea	(02) 1544 4564
South Eastern Asia	(65) 6277 3444

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GE Medical Systems Türkiye Ltd. Şti
Esentepe Mah. Harman Sok. No: 8
34394 Sisli-Istanbul
Türkiye

Addendum
0086 Infant Warmer System Heater Reflector Screws
Maintenance Check
Printed in USA
© 2016 General Electric Company

2098189-001
Revision A

Ohmeda Medical,
a Division of Datex-Ohmeda, Inc.,
a General Electric Company
8880 Gorman Road
Laurel MD 20723
USA