

DOCAPOST BPO IS - HILL ROM MAILING

Energy Park

150 Boulevard de Verdun

92400 Courbevoie – France



IDS MEDICAL SYSTEMS SINGAPORE PTE LTD

20 Science Park Road

Teletech Park \_01-23/25

117674 SINGAPORE

SINGAPOUR

---

## **Important Medical Device Correction**

**Subject: Progressa® Bed System with the P500 Microclimate Management™ Surface – Noisy Blower**

**FSCA-identifier:** MOD1240

**Type of action:** Correction

---

**Date:**

**To:** Chief Executive  
Facility Risk Manager  
Facility Administrator  
Facility Engineer  
Vigilance Manager  
Biomedical Engineering  
Medical Device Liaison Officer

**Affected devices:**

Models: Progressa® Bed (P7500) with P500 Microclimate Management™ Surface

Production Dates: Progressa® Beds manufactured prior to January 2015 containing APEX blowers

**Background:**

Hill-Rom has become aware of a potential issue with some Progressa® beds equipped with the integrated P500 Microclimate Management™ (MCM™) therapy surface. A bearing inside the MCM™ Surface Blower for the Progressa® bed has been prone to premature wearing, causing a high pitched sound to emanate from the MCM™ Surface Blower. This issue does not pose a hazard to the patient or affect normal functionality of the bed.

