



URGENT – Medical Device Correction Field Safety Notice

Philips V60 Ventilators May Be Impacted by Unresponsive Touchscreen

<Hospital/Clinic Name>

cc: Chairman Medical Board and relevant Head of Departments

Dear Customer,

A problem has been detected in certain Philips V60 Ventilators which could pose a risk for patients or users. This FSN86600042 is intended to inform you about:

- what the problem is and under what circumstances it can occur
- the actions that should be taken by the customer / user in order to prevent risks to patients or users
- the actions planned by Philips to correct the problem.

This document contains important information for the continued safe and proper use of your equipment

Please review the following information with all members of your staff who need to be aware of the contents of this communication. It is important to understand the implications of this communication.

Please retain a copy with the equipment Instruction for Use.

You can check the serial number of a V60 Ventilator to verify if it may be subject to this correction without pausing or discontinuing patient use. The following page includes directions on how to view the serial number of a V60 ventilator while it is in use.

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
If you need any further information or support concerning this issue, please contact your local Philips representative: Healthcare Representative/Modality Engineer: 1800-744-5477

Adverse reactions or quality problems experienced with the use of this product may be reported to the FDA's MedWatch Adverse Event Reporting program either online, by regular mail or by fax.

This notice has been reported to the appropriate Regulatory Agency.

Philips apologizes for any inconveniences caused by this problem.

Sincerely,

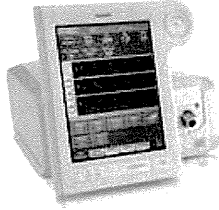
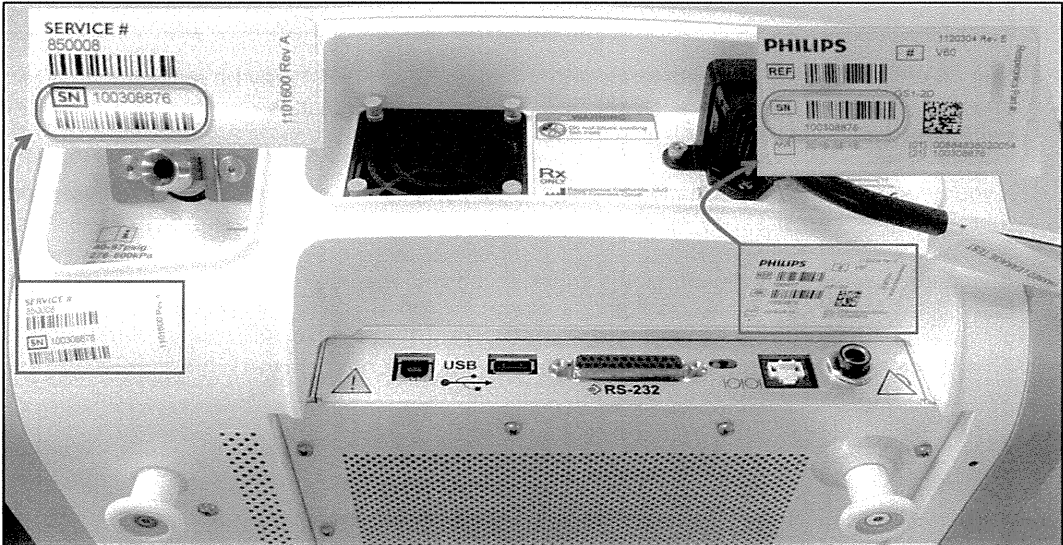

David McGrath
Head of Quality and Regulatory, HRC

PHILIPS

Therapeutic Care

-2/5-

FSN86600042A 30-Aug-2019

AFFECTED PRODUCTS	V60 Ventilators that have a serial number listed in the attachment. 
PROBLEM DESCRIPTION	The touchscreen must be used to change certain therapy settings. The touchscreens in affected units may become frozen and fail to respond to touch commands. Consequently, if a patient requires a change in a ventilator therapy setting, e.g., FiO₂, the change could not be made. Although it would be immediately apparent to the user that the changes cannot be made, there is no advance warning of touchscreen failure.
HAZARD INVOLVED	If the touchscreen on the V60 Ventilator malfunctions, therapy adjustment is delayed because the ventilator settings cannot be changed. A delay in therapy adjustment may lead to a drop in the patient's SpO₂ level and subsequently result in Hypercarbia and/or Hypoxemia. Note: The V60 ventilator will continue to function at the predetermined and accepted settings required to support the patient. In addition, patient settings, alarms, on-screen warnings, and waveforms are accurately displayed, visible, and alarms are annunciated.
HOW TO IDENTIFY AFFECTED PRODUCTS	Check the serial number of the ventilator against the attached list of affected units. Labeling containing device serial number information can be see located at the rear of the ventilator. (See Figure 1)  FIGURE 1: BACK VIEW OF V60 VENTILATOR

HOW TO IDENTIFY AFFECTED PRODUCTS

Alternatively, the serial number of the ventilator may also be accessed/reviewed from the display while the ventilator is in operation. Select the **Menu** tab at the bottom of the screen then select **Vent Info**. (See Figure 2 and 3)

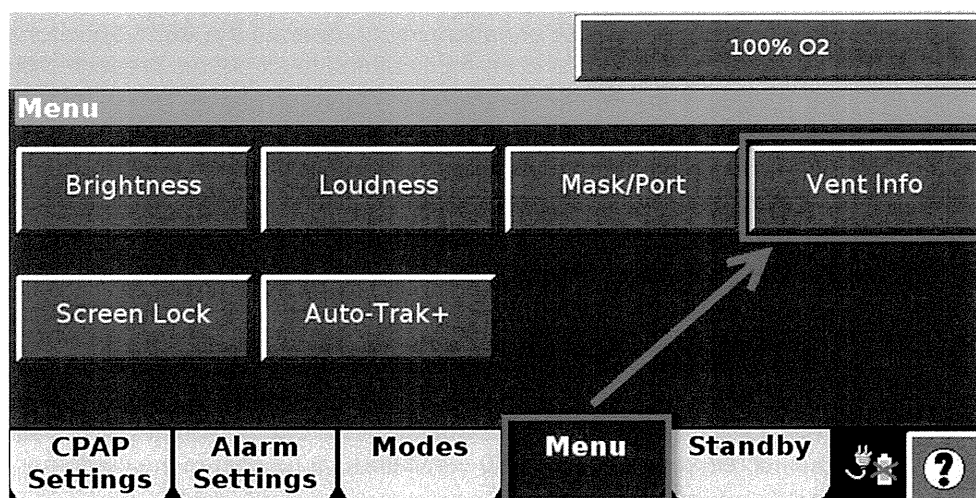


FIGURE 2: ACCESSING VENTILATOR SERIAL NUMBER FROM ON-SCREEN DISPLAY

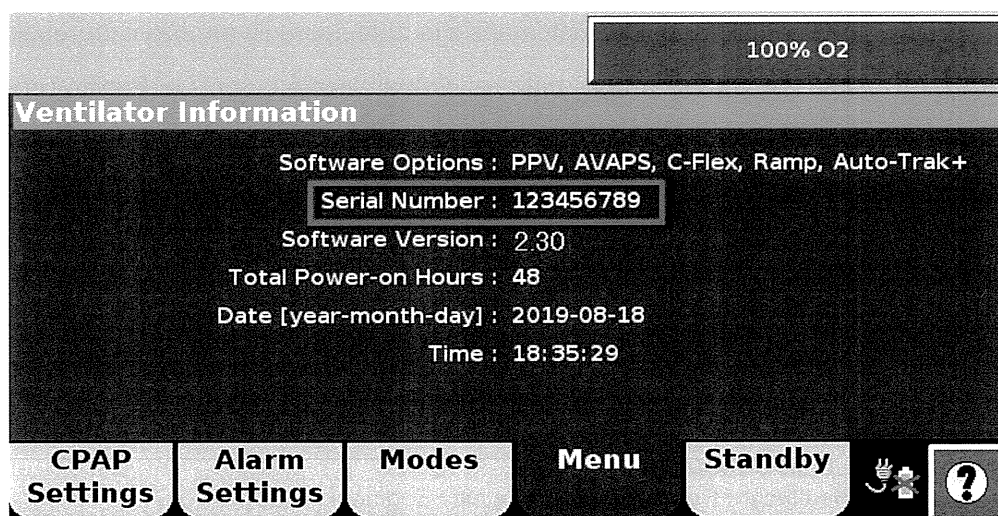


FIGURE 3: SERIAL NUMBER OF VENTILATOR AS VIEWED FROM THE VENT INFO SCREEN

PHILIPS

Therapeutic Care

-4/5-

FSN86600042A 30-Aug-2019

ACTION TO BE TAKEN BY CUSTOMER / USER	1) If in use on a patient, continue to use the ventilator if an alternative non-invasive ventilator is not available. 2) If a touchscreen failure is detected, disconnect the patient and immediately start ventilation with an alternate device. Contact your local customer service representative to report the failure. Please reference FCO86600042A. 3) To minimize the risk of patient injury, operate the V60 Ventilator as defined in the operator's manual and promptly attend to all alarms presented by the ventilator. 4) As recommended in the operator's manual, use an external O₂ monitor/analyzer and set the alarm thresholds appropriately. 5) As recommended in the operator's manual, have an alternative means of ventilation available whenever the ventilator is in use. 6) Acknowledge receipt of this notification 7) Receipt of this notification by any of the following methods: INSERT INFO HERE FOR THE APPROPRIATE MARKET
ACTIONS PLANNED BY PHILIPS	Philips will provide the following support for V60 customers by installing new touchscreens on affected V60 ventilators, at no cost to the customer. Philips Engineer or Philips Approved Service Provider Philips will contact each consignee to schedule an appointment to perform this correction. Philips Engineers or Philips Approved Service Providers will either repair any affected V60 ventilator at the customer's site or temporarily remove it for repair.
FURTHER INFORMATION AND SUPPORT	Monday through Friday between 8:00am and 5:00 pm US Pacific Time Firm responsible for this Medical Device Correction and Field Safety Notice: Respironics California, LLC 2271 Cosmos Court Carlsbad, CA 92011 Primary Contact Melissa Abbott Sr. Post Market Surveillance Manager Phone: +1 (760) 918-7300 E-mail: [REDACTED] Local Contact Healthcare Representative/Modality Engineer: 1800-744-5477



Therapeutic Care

-5/5-

FSN86600042A 30-Aug-2019

MEDICAL DEVICE FIELD SAFETY NOTICE RESPONSE
Philips V60 Ventilators may be Impacted by Unresponsive
Touchscreen

Acknowledgement and Receipt Form
Response is Required

Customer Information:

Form Completed by and Title:	
Contact Name and Title:	
Telephone Number:	
Email Address:	
Facility Name:	
Street Address:	
City, State, Zip Code:	
Country:	

I have read and understand the instructions provided in the notification letter. Yes ☐ No ☐

Have any adverse events associated with an unresponsive touchscreen occurred at your site? Yes ☐ No ☐

If yes, have you informed Philips of the event? Yes ☐ No ☐

If yes, please provide Philips Case Number _____ and if applicable, add any details below:

Details:

Signature: _____

Date: _____

Please return the completed and signed Acknowledgement and Receipt Form to: Healthcare Representative/Modality Engineer: 1800-744-5477

Row 1	Row 2	Row 3	Row 4	Row 5	Row 6	Row 7
100005249	100254792	100255723	100255865	100256311	100256502	100256609
100007779	100254793	100255724	100255866	100256312	100256503	100256610
100007982	100254977	100255725	100255965	100256313	100256504	100256611
100008795	100255098	100255726	100255966	100256314	100256505	100256612
100009556	100255165	100255727	100255967	100256315	100256507	100256613
100010513	100255317	100255728	100255968	100256316	100256508	100256614
100011641	100255318	100255729	100255969	100256317	100256509	100256615
100015789	100255409	100255730	100255970	100256318	100256510	100256616
100020831	100255421	100255732	100255971	100256319	100256511	100256617
100021459	100255422	100255733	100255972	100256395	100256512	100256618
100021751	100255424	100255734	100255973	100256396	100256513	100256619
100023634	100255500	100255737	100255974	100256398	100256514	100256620
100023729	100255501	100255738	100255975	100256399	100256515	100256621
100026949	100255502	100255739	100255976	100256400	100256516	100256623
100029508	100255504	100255840	100256005	100256401	100256517	100256624
100034162	100255506	100255841	100256127	100256402	100256518	100256625
100034395	100255507	100255842	100256128	100256403	100256519	100256626
100066214	100255509	100255843	100256129	100256404	100256520	100256627
100082340	100255510	100255844	100256130	100256405	100256521	100256628
100246732	100255512	100255845	100256131	100256406	100256522	100256629
100248301	100255513	100255846	100256132	100256407	100256523	100256630
100250369	100255547	100255847	100256133	100256408	100256563	100256631
100252951	100255548	100255848	100256134	100256410	100256564	100256632
100254466	100255556	100255849	100256135	100256411	100256565	100256633
100254552	100255558	100255850	100256136	100256412	100256566	100256634
100254738	100255594	100255851	100256137	100256413	100256567	100256635
100254739	100255596	100255852	100256138	100256415	100256568	100256636
100254745	100255597	100255853	100256139	100256416	100256569	100256637
100254746	100255598	100255854	100256300	100256417	100256570	100256638
100254747	100255599	100255855	100256301	100256418	100256571	100256639
100254748	100255620	100255856	100256302	100256419	100256572	100256640
100254749	100255621	100255857	100256303	100256420	100256573	100256641
100254750	100255622	100255858	100256304	100256422	100256574	100256642
100254751	100255623	100255859	100256305	100256423	100256575	100256643
100254752	100255626	100255860	100256306	100256424	100256576	100256644
100254783	100255676	100255861	100256307	100256425	100256577	100256645
100254785	100255678	100255862	100256308	100256426	100256578	100256646
100254787	100255721	100255863	100256309	100256500	100256607	100256647
100254790	100255722	100255864	100256310	100256501	100256608	100256780

Row 8	Row 9	Row 10	Row 11	Row 12	Row 13	Row 14
100256781	100256875	100257173	100257312	100257583	100257698	100258034
100256782	100256876	100257174	100257313	100257584	100257699	100258035
100256783	100256877	100257175	100257314	100257585	100257800	100258159
100256784	100256980	100257176	100257316	100257586	100257801	100258211
100256785	100256981	100257177	100257317	100257587	100257802	100258240
100256786	100256982	100257178	100257318	100257588	100257803	100258241
100256842	100256983	100257179	100257319	100257589	100257804	100258242
100256843	100256984	100257200	100257377	100257590	100257805	100258243
100256844	100256985	100257201	100257378	100257591	100257806	100258244
100256845	100256986	100257202	100257379	100257592	100257807	100258245
100256846	100256987	100257203	100257380	100257593	100257808	100258246
100256847	100256988	100257204	100257381	100257670	100257809	100258247
100256848	100256990	100257205	100257382	100257671	100257811	100258248
100256849	100256991	100257206	100257383	100257672	100257812	100258249
100256850	100257085	100257207	100257384	100257673	100257813	100258250
100256851	100257086	100257208	100257385	100257674	100257814	100258272
100256852	100257087	100257209	100257386	100257675	100257815	100258273
100256853	100257089	100257210	100257387	100257676	100257816	100258274
100256854	100257090	100257211	100257388	100257677	100257817	100258275
100256855	100257091	100257214	100257389	100257678	100257818	100258276
100256856	100257092	100257215	100257390	100257679	100257819	100258277
100256857	100257093	100257216	100257391	100257680	100257921	100258278
100256858	100257094	100257217	100257392	100257681	100257922	100258279
100256859	100257095	100257218	100257393	100257682	100257923	100258280
100256860	100257096	100257219	100257394	100257683	100257924	100258281
100256861	100257097	100257260	100257395	100257684	100257925	100258282
100256862	100257098	100257261	100257396	100257685	100258020	100258283
100256863	100257099	100257262	100257397	100257686	100258021	100258284
100256864	100257162	100257263	100257398	100257687	100258022	100258285
100256865	100257163	100257264	100257399	100257688	100258023	100258286
100256866	100257164	100257265	100257400	100257689	100258024	100258289
100256867	100257165	100257266	100257401	100257690	100258025	100258290
100256868	100257166	100257267	100257403	100257691	100258026	100258291
100256869	100257167	100257268	100257404	100257692	100258027	100258292
100256870	100257168	100257269	100257405	100257693	100258028	100258293
100256871	100257169	100257270	100257406	100257694	100258029	100258295
100256872	100257170	100257271	100257580	100257695	100258030	100258297
100256873	100257171	100257310	100257581	100257696	100258031	100258298
100256874	100257172	100257311	100257582	100257697	100258032	100258299

Row 15	Row 16	Row 17	Row 18	Row 19	Row 20	Row 21
100258360	100258513	100258809	100259082	100259249	100259633	100259878
100258361	100258514	100258810	100259083	100259250	100259634	100259879
100258362	100258553	100258811	100259085	100259251	100259635	100259880
100258363	100258554	100258812	100259086	100259252	100259636	100259881
100258364	100258556	100258813	100259087	100259253	100259724	100259882
100258365	100258557	100258814	100259088	100259255	100259726	100259996
100258366	100258558	100258817	100259089	100259256	100259732	100259997
100258367	100258559	100258818	100259090	100259258	100259735	100259998
100258368	100258587	100258819	100259091	100259259	100259737	100259999
100258450	100258588	100258820	100259092	100259260	100259783	100260020
100258451	100258589	100258821	100259093	100259261	100259784	100260021
100258452	100258590	100258822	100259094	100259262	100259785	100260022
100258453	100258591	100258823	100259095	100259263	100259786	100260024
100258454	100258592	100258920	100259096	100259380	100259788	100260025
100258455	100258593	100258921	100259097	100259381	100259791	100260026
100258456	100258594	100258922	100259098	100259382	100259792	100260027
100258457	100258596	100258923	100259099	100259383	100259794	100260028
100258471	100258597	100258924	100259100	100259384	100259796	100260030
100258472	100258598	100258925	100259102	100259385	100259797	100260031
100258473	100258600	100258926	100259103	100259386	100259853	100260032
100258474	100258601	100258927	100259135	100259387	100259854	100260033
100258475	100258602	100258928	100259136	100259388	100259858	100260034
100258476	100258603	100258929	100259137	100259389	100259860	100260035
100258477	100258604	100258930	100259138	100259390	100259862	100260036
100258478	100258606	100258931	100259139	100259391	100259863	100260037
100258479	100258607	100258932	100259234	100259392	100259864	100260038
100258495	100258609	100258933	100259235	100259393	100259865	100260039
100258496	100258610	100258934	100259236	100259394	100259866	100260040
100258497	100258611	100258935	100259237	100259395	100259867	100260042
100258499	100258613	100258936	100259238	100259396	100259868	100260043
100258500	100258614	100258937	100259239	100259397	100259869	100260044
100258501	100258800	100258938	100259240	100259398	100259870	100260045
100258505	100258801	100258939	100259241	100259399	100259871	100260174
100258506	100258802	100258941	100259242	100259400	100259872	100260175
100258507	100258803	100258942	100259243	100259401	100259873	100260177
100258508	100258805	100259056	100259244	100259402	100259874	100260179
100258509	100258806	100259057	100259245	100259403	100259875	100260180
100258510	100258807	100259059	100259246	100259404	100259876	100260182
100258511	100258808	100259081	100259248	100259632	100259877	100260183

Row 22	Row 23	Row 24	Row 25	Row 26	Row 27	Row 28
100260186	100260534	100260846	100261115	100261278	100261538	100261743
100260231	100260535	100260847	100261117	100261279	100261539	100261744
100260233	100260537	100260849	100261120	100261302	100261540	100261745
100260234	100260538	100260850	100261121	100261320	100261541	100261746
100260235	100260539	100260851	100261122	100261321	100261543	100261747
100260236	100260540	100260852	100261123	100261322	100261544	100261748
100260237	100260541	100260853	100261124	100261365	100261545	100261749
100260238	100260542	100260854	100261125	100261366	100261546	100261750
100260239	100260545	100260855	100261126	100261367	100261547	100261751
100260240	100260546	100260856	100261127	100261368	100261549	100261752
100260241	100260547	100260857	100261128	100261369	100261550	100261753
100260242	100260548	100260858	100261129	100261370	100261551	100261754
100260243	100260549	100260951	100261130	100261371	100261552	100261755
100260244	100260550	100260953	100261131	100261372	100261553	100261756
100260245	100260551	100260955	100261132	100261373	100261554	100261757
100260246	100260552	100260956	100261133	100261374	100261555	100261758
100260247	100260553	100260957	100261134	100261375	100261556	100261759
100260248	100260554	100260958	100261135	100261376	100261557	100262080
100260249	100260708	100260960	100261136	100261378	100261558	100262082
100260251	100260710	100260961	100261137	100261379	100261559	100262083
100260252	100260711	100260962	100261138	100261400	100261560	100262084
100260253	100260712	100260963	100261139	100261401	100261561	100262085
100260254	100260713	100260964	100261140	100261402	100261562	100262086
100260255	100260714	100260965	100261168	100261404	100261563	100262087
100260256	100260715	100260966	100261169	100261406	100261564	100262088
100260257	100260831	100260967	100261170	100261407	100261667	100262089
100260259	100260832	100260968	100261171	100261408	100261668	100262090
100260440	100260833	100260970	100261172	100261409	100261669	100262131
100260451	100260834	100260971	100261173	100261410	100261670	100262132
100260452	100260835	100260972	100261174	100261411	100261671	100262133
100260525	100260836	100260973	100261175	100261412	100261672	100262134
100260526	100260837	100260974	100261176	100261413	100261673	100262135
100260527	100260838	100261002	100261177	100261414	100261674	100262136
100260528	100260839	100261003	100261178	100261452	100261675	100262137
100260529	100260840	100261004	100261179	100261453	100261676	100262138
100260530	100260841	100261111	100261273	100261454	100261677	100262139
100260531	100260842	100261112	100261274	100261455	100261678	100262167
100260532	100260843	100261113	100261275	100261456	100261679	100262168
100260533	100260845	100261114	100261276	100261537	100261742	100262169

Row 29	Row 30	Row 31	Row 32	Row 33	Row 34	Row 35
100262170	100262417	100262531	100262627	100262821	100262964	100263281
100262171	100262418	100262532	100262628	100262822	100262965	100263282
100262172	100262419	100262533	100262629	100262823	100262966	100263283
100262173	100262420	100262534	100262630	100262824	100262967	100263284
100262174	100262421	100262535	100262631	100262825	100262968	100263285
100262175	100262422	100262536	100262632	100262826	100262969	100263286
100262176	100262423	100262537	100262633	100262827	100262970	100263287
100262177	100262424	100262539	100262634	100262828	100262971	100263288
100262178	100262426	100262540	100262635	100262829	100262972	100263289
100262179	100262427	100262541	100262637	100262830	100262973	100263290
100262180	100262429	100262543	100262638	100262832	100262975	100263291
100262181	100262431	100262544	100262639	100262833	100263120	100263292
100262182	100262432	100262546	100262640	100262834	100263121	100263293
100262183	100262433	100262547	100262641	100262835	100263122	100263294
100262184	100262434	100262548	100262642	100262836	100263123	100263295
100262185	100262436	100262549	100262750	100262837	100263124	100263296
100262186	100262437	100262550	100262751	100262839	100263125	100263297
100262187	100262438	100262551	100262753	100262940	100263126	100263298
100262188	100262439	100262552	100262754	100262942	100263128	100263299
100262189	100262440	100262603	100262755	100262943	100263129	100263300
100262190	100262441	100262604	100262756	100262944	100263130	100263301
100262191	100262507	100262605	100262757	100262945	100263131	100263302
100262192	100262508	100262606	100262758	100262946	100263132	100263303
100262193	100262510	100262608	100262759	100262947	100263133	100263304
100262194	100262511	100262610	100262805	100262949	100263134	100263305
100262195	100262512	100262611	100262806	100262950	100263135	100263306
100262196	100262513	100262612	100262807	100262951	100263136	100263307
100262198	100262514	100262613	100262808	100262952	100263137	100263308
100262199	100262515	100262616	100262809	100262953	100263139	100263309
100262200	100262520	100262617	100262810	100262954	100263189	100263310
100262201	100262522	100262618	100262811	100262955	100263190	100263311
100262202	100262523	100262619	100262812	100262956	100263191	100263312
100262203	100262524	100262620	100262813	100262957	100263193	100263313
100262204	100262525	100262621	100262814	100262958	100263194	100263315
100262205	100262526	100262622	100262815	100262959	100263195	100263535
100262206	100262527	100262623	100262816	100262960	100263196	100263536
100262207	100262528	100262624	100262817	100262961	100263197	100263537
100262209	100262529	100262625	100262818	100262962	100263198	100263538
100262210	100262530	100262626	100262819	100262963	100263199	100263539
					100263280	100263540

Row 36	Row 37	Row 38	Row 39	Row 40	Row 41	Row 42
100263541	100263659	100263704	100264442	100265434	100265700	100266018
100263542	100263660	100263705	100264443	100265435	100265701	100266020
100263543	100263661	100263706	100264444	100265436	100265702	100266021
100263544	100263662	100263708	100264445	100265437	100265703	100266022
100263545	100263663	100263709	100264581	100265438	100265704	100266023
100263546	100263664	100264295	100264582	100265439	100265705	100266024
100263547	100263665	100264297	100264583	100265459	100265706	100266025
100263548	100263666	100264298	100264584	100265460	100265708	100266026
100263549	100263667	100264299	100264585	100265461	100265820	100266027
100263550	100263668	100264406	100264586	100265462	100265821	100266028
100263551	100263669	100264407	100264587	100265463	100265822	100266029
100263552	100263670	100264408	100264588	100265464	100265823	100266030
100263555	100263671	100264410	100264589	100265465	100265824	100266031
100263556	100263672	100264411	100264590	100265466	100265825	100266033
100263557	100263673	100264412	100264591	100265467	100265826	100266034
100263558	100263674	100264413	100264592	100265468	100265827	100266035
100263559	100263675	100264414	100264593	100265469	100265828	100266036
100263561	100263676	100264415	100264595	100265470	100265829	100266037
100263562	100263677	100264416	100264596	100265471	100265831	100266038
100263563	100263678	100264417	100264597	100265472	100265832	100266039
100263564	100263680	100264418	100264598	100265679	100265834	100266040
100263565	100263682	100264420	100264599	100265680	100265835	100266041
100263566	100263683	100264421	100264600	100265682	100265836	100266042
100263567	100263684	100264423	100264601	100265683	100265837	100266043
100263568	100263685	100264424	100264602	100265684	100265838	100266140
100263569	100263686	100264425	100264603	100265685	100265839	100266141
100263570	100263687	100264426	100264604	100265686	100265840	100266142
100263571	100263688	100264427	100264605	100265687	100265841	100266143
100263572	100263689	100264428	100264606	100265688	100265842	100266144
100263573	100263690	100264429	100264607	100265689	100265843	100266145
100263574	100263691	100264431	100264608	100265690	100265844	100266146
100263575	100263692	100264432	100264609	100265691	100265845	100266147
100263576	100263693	100264433	100264610	100265692	100265846	100266148
100263577	100263694	100264434	100264611	100265693	100265847	100266149
100263578	100263696	100264435	100264612	100265694	100265848	100266150
100263579	100263697	100264436	100264613	100265695	100265849	100266151
100263580	100263698	100264437	100265429	100265696	100266014	100266152
100263582	100263699	100264439	100265430	100265697	100266015	100266153
100263583	100263701	100264440	100265432	100265698	100266016	100266154
100263584	100263703	100264441	100265433	100265699	100266017	100266155

Row 43	Row 44	Row 45	Row 46	Row 47	Row 48	Row 49
100266156	100266267	100266523	100266687	100266874	100267094	100267311
100266157	100266369	100266524	100266688	100266875	100267095	100267312
100266158	100266370	100266525	100266689	100266876	100267096	100267313
100266159	100266371	100266526	100266690	100266877	100267097	100267314
100266160	100266372	100266527	100266691	100266878	100267098	100267315
100266161	100266374	100266528	100266692	100266879	100267099	100267316
100266163	100266375	100266529	100266693	100266900	100267120	100267317
100266164	100266376	100266530	100266694	100266901	100267121	100267318
100266165	100266377	100266531	100266742	100266902	100267122	100267319
100266166	100266378	100266532	100266743	100266903	100267123	100267320
100266167	100266379	100266533	100266744	100266904	100267124	100267321
100266168	100266400	100266534	100266745	100266905	100267125	100267322
100266198	100266401	100266535	100266746	100266906	100267126	100267323
100266199	100266402	100266536	100266747	100266907	100267127	100267324
100266240	100266403	100266537	100266748	100266908	100267128	100267325
100266241	100266404	100266538	100266749	100266909	100267129	100267326
100266242	100266405	100266539	100266750	100266910	100267130	100267327
100266243	100266406	100266540	100266751	100266911	100267131	100267328
100266244	100266407	100266541	100266752	100266912	100267132	100267419
100266245	100266408	100266542	100266753	100266913	100267133	100267420
100266246	100266409	100266619	100266754	100266914	100267134	100267421
100266247	100266410	100266665	100266755	100266915	100267135	100267422
100266248	100266411	100266666	100266756	100266916	100267136	100267423
100266249	100266412	100266667	100266757	100266917	100267137	100267424
100266250	100266413	100266668	100266758	100266918	100267138	100267425
100266251	100266414	100266669	100266759	100266919	100267139	100267426
100266252	100266415	100266670	100266760	100266920	100267140	100267427
100266253	100266416	100266672	100266761	100266921	100267141	100267428
100266254	100266417	100266673	100266762	100267082	100267142	100267429
100266255	100266418	100266674	100266763	100267083	100267300	100267430
100266257	100266513	100266675	100266764	100267084	100267301	100267431
100266258	100266514	100266676	100266765	100267085	100267302	100267432
100266259	100266515	100266677	100266766	100267086	100267303	100267433
100266260	100266516	100266678	100266767	100267087	100267304	100267434
100266261	100266517	100266679	100266768	100267088	100267305	100267435
100266262	100266518	100266680	100266769	100267089	100267306	100267436
100266263	100266519	100266682	100266770	100267090	100267307	100267437
100266264	100266520	100266683	100266771	100267091	100267308	100267438
100266265	100266521	100266685	100266872	100267092	100267309	100267439
100266266	100266522	100266686	100266873	100267093	100267310	100267440

Row 50	Row 51	Row 52	Row 53	Row 54	Row 55	Row 56
100267441	100267573	100267848	100267989	100268193	100268332	100268500
100267442	100267574	100267849	100267990	100268194	100268333	100268501
100267443	100267688	100267850	100267991	100268195	100268334	100268502
100267444	100267689	100267851	100267992	100268244	100268335	100268503
100267445	100267690	100267852	100267993	100268245	100268337	100268504
100267446	100267691	100267853	100267994	100268246	100268338	100268505
100267447	100267692	100267854	100267995	100268247	100268339	100268506
100267448	100267693	100267855	100267996	100268248	100268426	100268507
100267449	100267694	100267856	100267997	100268249	100268427	100268508
100267450	100267695	100267857	100267998	100268250	100268428	100268509
100267451	100267696	100267859	100268000	100268251	100268429	100268510
100267452	100267697	100267860	100268164	100268252	100268430	100268511
100267453	100267698	100267861	100268165	100268253	100268431	100268512
100267525	100267699	100267862	100268166	100268254	100268432	100268513
100267526	100267700	100267863	100268167	100268255	100268433	100268600
100267527	100267701	100267864	100268168	100268256	100268434	100268601
100267528	100267702	100267865	100268169	100268258	100268435	100268602
100267529	100267703	100267966	100268170	100268259	100268436	100268603
100267531	100267704	100267967	100268171	100268263	100268437	100268604
100267532	100267705	100267968	100268172	100268264	100268438	100268605
100267533	100267706	100267969	100268173	100268265	100268439	100268606
100267534	100267707	100267970	100268174	100268266	100268480	100268607
100267535	100267708	100267971	100268175	100268267	100268481	100268608
100267536	100267811	100267972	100268176	100268268	100268482	100268609
100267537	100267812	100267973	100268177	100268269	100268483	100268610
100267538	100267813	100267974	100268178	100268270	100268485	100268611
100267539	100267814	100267975	100268179	100268271	100268486	100268612
100267560	100267815	100267976	100268180	100268272	100268487	100268613
100267561	100267836	100267977	100268181	100268320	100268488	100268614
100267562	100267837	100267978	100268182	100268321	100268489	100268615
100267563	100267838	100267979	100268183	100268322	100268490	100268616
100267564	100267839	100267980	100268184	100268323	100268491	100268617
100267565	100267840	100267981	100268185	100268324	100268492	100268618
100267566	100267841	100267982	100268186	100268325	100268493	100268619
100267567	100267842	100267983	100268187	100268326	100268494	100268620
100267568	100267843	100267984	100268188	100268327	100268495	100268621
100267569	100267844	100267985	100268189	100268328	100268496	100268622
100267570	100267845	100267986	100268190	100268329	100268497	100268623
100267571	100267846	100267987	100268191	100268330	100268498	100268624
100267572	100267847	100267988	100268192	100268331	100268499	100268625

Row 57	Row 58	Row 59	Row 60	Row 61	Row 62	Row 63
100268626	100268894	100269107	100269155	100269326	100269611	100269781
100268627	100268895	100269108	100269156	100269327	100269612	100269782
100268628	100268896	100269109	100269157	100269328	100269613	100269783
100268629	100268897	100269110	100269158	100269329	100269614	100269784
100268630	100268911	100269111	100269159	100269330	100269615	100269785
100268631	100268912	100269112	100269160	100269331	100269616	100269786
100268860	100268913	100269113	100269161	100269332	100269617	100269787
100268861	100268914	100269115	100269162	100269333	100269618	100269788
100268862	100268915	100269116	100269163	100269334	100269619	100269789
100268863	100268916	100269117	100269164	100269335	100269621	100269790
100268864	100268917	100269118	100269165	100269336	100269622	100269791
100268865	100268918	100269119	100269166	100269582	100269623	100269792
100268866	100268919	100269127	100269167	100269583	100269624	100269793
100268867	100269080	100269128	100269168	100269584	100269625	100269794
100268868	100269081	100269129	100269169	100269585	100269626	100269795
100268869	100269082	100269130	100269170	100269586	100269627	100269796
100268870	100269083	100269131	100269300	100269587	100269628	100269797
100268871	100269084	100269132	100269301	100269588	100269629	100269798
100268872	100269085	100269133	100269302	100269589	100269630	100269799
100268873	100269086	100269134	100269303	100269590	100269631	100269800
100268874	100269087	100269135	100269304	100269591	100269632	100269801
100268875	100269088	100269136	100269305	100269592	100269633	100269802
100268876	100269089	100269137	100269306	100269593	100269634	100269803
100268877	100269090	100269138	100269307	100269594	100269635	100269804
100268878	100269091	100269139	100269308	100269595	100269636	100269805
100268879	100269092	100269140	100269309	100269596	100269637	100269806
100268880	100269093	100269141	100269310	100269597	100269638	100269807
100268881	100269094	100269142	100269311	100269598	100269768	100269808
100268882	100269095	100269143	100269312	100269599	100269769	100269809
100268883	100269096	100269144	100269313	100269600	100269770	100269810
100268884	100269097	100269145	100269314	100269601	100269771	100269811
100268885	100269098	100269146	100269315	100269602	100269772	100269812
100268886	100269099	100269147	100269316	100269603	100269773	100269813
100268887	100269100	100269148	100269317	100269604	100269774	100269814
100268888	100269101	100269149	100269318	100269605	100269775	100269815
100268889	100269102	100269150	100269319	100269606	100269776	100269816
100268890	100269103	100269151	100269320	100269607	100269777	100269817
100268891	100269104	100269152	100269321	100269608	100269778	100269889
100268892	100269105	100269153	100269322	100269609	100269779	100269890
100268893	100269106	100269154	100269323	100269610	100269780	100269891

Row 64	Row 65	Row 66	Row 67	Row 68	Row 69	Row 70
100269892	100270033	100270287	100270485	100270774	100270968	100271029
100269893	100270034	100270288	100270486	100270775	100270969	100271030
100269894	100270035	100270289	100270487	100270776	100270970	100271031
100269895	100270036	100270290	100270488	100270777	100270971	100271032
100269896	100270037	100270291	100270660	100270778	100270972	100271033
100269897	100270038	100270292	100270661	100270779	100270973	100271034
100269898	100270039	100270293	100270662	100270780	100270974	100271035
100269899	100270040	100270294	100270663	100270781	100270975	100271036
100270001	100270252	100270295	100270664	100270782	100270976	100271037
100270002	100270254	100270296	100270665	100270783	100270977	100271038
100270003	100270255	100270297	100270666	100270784	100270978	100271039
100270004	100270256	100270298	100270667	100270785	100270979	100271113
100270005	100270258	100270299	100270668	100270786	100270980	100271114
100270006	100270259	100270300	100270669	100270787	100270981	100271115
100270007	100270260	100270301	100270670	100270788	100270982	100271116
100270008	100270261	100270460	100270671	100270789	100270983	100271117
100270009	100270262	100270461	100270672	100270790	100270984	100271118
100270010	100270263	100270462	100270673	100270791	100270985	100271119
100270011	100270264	100270463	100270674	100270792	100270986	100271180
100270012	100270265	100270464	100270675	100270793	100270987	100271181
100270013	100270266	100270465	100270676	100270794	100270988	100271182
100270014	100270267	100270466	100270677	100270795	100270989	100271183
100270015	100270268	100270467	100270678	100270796	100270990	100271184
100270016	100270269	100270468	100270679	100270797	100270991	100271185
100270017	100270270	100270469	100270680	100270798	100270992	100271186
100270018	100270271	100270470	100270681	100270799	100270994	100271187
100270019	100270272	100270471	100270682	100270800	100270995	100271188
100270020	100270273	100270472	100270683	100270801	100270996	100271189
100270021	100270274	100270473	100270684	100270802	100270997	100271190
100270022	100270275	100270474	100270685	100270804	100270998	100271191
100270023	100270277	100270475	100270686	100270805	100270999	100271192
100270024	100270278	100270476	100270687	100270806	100271000	100271193
100270025	100270279	100270477	100270688	100270807	100271001	100271313
100270026	100270280	100270478	100270689	100270808	100271022	100271314
100270027	100270281	100270479	100270690	100270962	100271023	100271315
100270028	100270282	100270480	100270769	100270963	100271024	100271316
100270029	100270283	100270481	100270770	100270964	100271025	100271317
100270030	100270284	100270482	100270771	100270965	100271026	100271318
100270031	100270285	100270483	100270772	100270966	100271027	100271319
100270032	100270286	100270484	100270773	100270967	100271028	100271320

Row 71	Row 72	Row 73	Row 74	Row 75	Row 76	Row 77
100271321	100271586	100271738	201035373	201035458	201035506	201035563
100271322	100271587	100271739	201035374	201035459	201035507	201035564
100271323	100271588	100271800	201035395	201035460	201035508	201035565
100271324	100271590	100271801	201035396	201035461	201035509	201035566
100271325	100271591	100271802	201035397	201035462	201035510	201035567
100271326	100271592	100271803	201035398	201035463	201035511	201035568
100271327	100271593	100271804	201035399	201035464	201035512	201035569
100271328	100271594	100271805	201035423	201035465	201035513	201035570
100271329	100271595	100271806	201035424	201035466	201035514	201035571
100271330	100271596	100271807	201035425	201035467	201035515	201035572
100271331	100271597	100271808	201035426	201035469	201035516	201035573
100271332	100271598	100271809	201035428	201035470	201035517	201035574
100271333	100271599	100271905	201035430	201035471	201035518	201035575
100271334	100271660	100271906	201035431	201035472	201035519	201035576
100271335	100271661	100271907	201035432	201035473	201035520	201035577
100271336	100271662	100271908	201035433	201035474	201035521	201035578
100271337	100271663	100271909	201035434	201035475	201035522	
100271338	100271664	100271910	201035435	201035476	201035540	
100271339	100271665	201001641	201035436	201035477	201035541	
100271340	100271666	201001920	201035437	201035478	201035542	
100271341	100271667	201002700	201035438	201035479	201035543	
100271342	100271668	201002716	201035439	201035480	201035544	
100271343	100271669	201034540	201035440	201035481	201035545	
100271344	100271670	201034549	201035441	201035482	201035546	
100271520	100271671	201035299	201035442	201035483	201035547	
100271521	100271723	201035310	201035443	201035484	201035548	
100271522	100271724	201035311	201035444	201035485	201035549	
100271523	100271725	201035312	201035445	201035486	201035550	
100271525	100271726	201035313	201035446	201035487	201035551	
100271526	100271727	201035314	201035447	201035488	201035552	
100271527	100271728	201035315	201035448	201035489	201035553	
100271528	100271729	201035316	201035449	201035490	201035554	
100271529	100271730	201035317	201035450	201035491	201035555	
100271530	100271731	201035318	201035451	201035492	201035556	
100271531	100271732	201035319	201035452	201035500	201035557	
100271532	100271733	201035368	201035453	201035501	201035558	
100271533	100271734	201035369	201035454	201035502	201035559	
100271534	100271735	201035370	201035455	201035503	201035560	
100271535	100271736	201035371	201035456	201035504	201035561	
100271536	100271737	201035372	201035457	201035505	201035562	