



27 Jul 2018

Product: Panocell®-10

LOT 22318

2018-08-03

REF 0003032

UDI

Product Notification

Issue:

Immucor is issuing a product notification regarding the V antigen typing for donor D2022, vial 4 of Panocell-10, lot 22318. Additional DNA testing of donor D2022 has confirmed the donor as V-, instead of V+ as indicated on the Master List.

Impact to Results:

Since donor D2022 is V-, the red blood cells in vial 4 will be nonreactive with specimens that may contain anti-V. V (RH10) is a low prevalence antigen in the RH system. You can evaluate the impact on any previous testing results using these lots per your procedures.

Required Action:

A corrected Master List for Panocell-10, lot 22318 is provided with this notification and is identified with an "A" in the **upper-left corner**. Please replace your original Master List with this corrected Master List. An updated Supplemental Unconfirmed Typings indicating donor D2022 as DNA typed has also been provided.

Please acknowledge your receipt of this notification by completing the response form below and returning it to **Cherry Teh** by **31 July 2018**.

If you have questions or require additional information, please contact [REDACTED]

We apologize for the inconvenience this issue may have caused your laboratory.

Sincerely,

Cherry Teh
Business Executive

RESPONSE FORM

I acknowledge that our facility is aware of the product notification for Panocell-10, lot 22318 (FA-NOR-18-06)	
Name:	[REDACTED]
Position:	[REDACTED]
Facility:	[REDACTED]
Account number:	[REDACTED]
Address:	
Telephone:	

3130 Gateway Drive
Norcross, GA 30071
855.466.8267
www.immucor.com