

## **URGENT    FIELD SAFETY NOTICE**

**No. : FCO-XR-18-0014Z**  
**Control No. : QAX 19-028**  
**Type of Action : Recall**  
**June 2019**

### **Information about addition of finger pinching prevention protective cover**

---

**Dear customer ,**

This document contains important information for the continued safe and proper use of your Hitachi X-ray System.

**Details of Affected Devices :**

Product Name    : CLINIX II (YTS-1N)

**Description of problem :**

- 1. An incident of a injury of finger being pinched between the tabletop and retaining grid under the tabletop has occurred in the radiography stand YTS-1N integrated with the floor travel support device that is a component of the X-ray radiography equipment CLINIX series.
- 2. It may occur.
- 3. Injury of finger being pinched between the tabletop and retaining grid under the tabletop.

**Advise on action to be taken by the users:**

When a patient goes down from a table, please be careful not to pinch the finger between tabletop and retaining grid under the tabletop.

**Transmission of this Field Safety Notice:**

This notice needs to be passed on all those who need to be aware within your organization or to any organization where the potentially affected device has been transferred.

**Geographic Distribution :**

Japan , Taiwan , Indonesia , Thailand

**Contact reference person:**

Please find your local contact person on the cover letter of our local sales organization to this "Urgent Field Safety Notice".

We confirm that this item has been reported to the responsible National Competent Authority.

Yours sincerely  
Hitachi Ltd.

---

Rev.1.7