



Dear Healthcare Professional,

Voluntary Recall of Nova T Copper Intrauterine Device (DE0002264)

Please be informed about a voluntary recall of specific batches of Nova T copper IUD (T-shaped Copper IUD model with 200 mm² copper surface) by Bayer.

Affected Batches are:

TU00TP5 S1
TU00VV5 S1
TU00ZNV S1
TU012AX S1
TU013U9 S1

An increased device breakage rate for Nova T 200 IUD was observed. The breakage was noticed during insertion or shortly after insertion in connection with expulsion of the T-body loop with attached threads.

The root cause investigation showed that the increased breakage rate can be traced back to a change in the T-body mold (due to a supplier change) in combination with the use of a specific raw material batch. This caused a weakness in the T-body of Nova T 200 above the loop to which the removal threads are attached. It leads to an increased risk for breakage when the T-body is bent sideways, or when the strings are pulled at an angle.

The mechanism of breakage identified during the investigations, and the typical pattern of breakage observed in the cases reported to date indicate that the increased risk of breakage is linked to the insertion (possible bending force when pushing the T-body out of the insertion tube).

Based on the currently available data and nature of the breakage, no impact on the efficacy of Nova T 200 is expected.

23 Dec 2015

Bayer (South East Asia) Pte Ltd
BHC Pharmaceuticals
63 Chulia Street
OCBC Centre East, 14th Floor
Singapore 049514

Tel : +65 6496 1888
Fax: +65 6496 1498

www.bayer.com

Co. No.: 197901344Z



Page 2

Other IUDs by Bayer (e.g. Mirena, Jaydess, Nova T 380) are not affected by the detected defect as described above. In Singapore, only Mirena and Nova T 200 are registered.

The stocks of the affected Nova T 200 batches are blocked by Bayer. The affected batches are recalled from all distribution channels by Bayer, the affected batches shall be sent back to our distributor, Zuellig Pharma Pte Ltd. As of date, there are no reports of similar defect in Singapore.

Advice on Action to be taken:

- Further insertion of IUDs of the affected batches should not be performed.
- The removal of an already successfully inserted IUD in absence of any symptom or any other reason for removal is not considered justified in view of the potential risk associated with an additional removal and insertion of a new IUD.
- Planned removals of Nova T 200 should be performed by grasping the removal threads with a forceps and applying gentle traction, as per routine procedure.

Recall Action:

If the affected batches are present in your inventory, please kindly contact your respective sales representatives before 28th December 2015.

Transmission of this Field Safety Notice:

This notice needs to be passed on to all those who need to be aware within your organization, or to any organisation, where the potentially affected devices have been transferred to.

Please transfer this notice to other organisations on which this action has an impact.

Please maintain awareness of this notice and resulting action for an appropriate period of time to ensure effectiveness of the corrective action.




Page 3

Contact Person:


If you have any query pertaining to the above matter, please contact Bayer (South East Asia) Pte Ltd at Tel: 65-6496-1922.

Email: medinfo.singapore@bayer.com

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Yours sincerely,
Bayer (South East Asia) Pte Ltd



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Dr. Shivani Kapur
Country Medical Director,
Singapore, Malaysia & Brunei



Yamini Prakash
Business Unit Head
Specialty Medicine