

████████████████████
 ████████████████████
 ████████████████████
 ████████████████████

14 Feb 2019

Urgent FIELD SAFETY NOTICE**IV Pole Collar for Spectra Optia® Apheresis System**

Name: Spectra Optia® Apheresis System
 Applicable Product codes: 61000

Dear Valued Customer,

Terumo BCT is dedicated to providing you with the highest quality support and remains committed to communicating information regarding Terumo BCT products. This letter provides information about the new IV pole collar accessory for the Spectra Optia system.

New Spectra Optia systems will be shipped with an IV pole collar (shown in Figure 1 below). This feature has been added to avoid a pinch point between the IV pole and the machine while lowering the IV pole.

If the operator does not hold onto the IV pole while pressing the IV pole release button, the IV pole may drop in a free fall. If the operator's hand or any other body part is in the path of the free-falling IV pole, the operator could be injured. Because this is an added safety feature, do not remove the collars after they are installed on the IV pole.

Your local Terumo BCT representatives will work with you to ensure IV pole collars are provided



Figure 1: Photo of the IV pole collar installed on the Spectra Optia IV pole.



Figure 2: Photo of the IV pole collar prior to installation.

and installed on Spectra Optia systems already in the field. On new machines, service technicians will attach the IV pole collar to the IV pole during installation.

Terumo BCT, Inc.
 10811 West Collins Ave.
 Lakewood, Colorado 80215-4440
 USA
 USA Phone: 1.877.339.4228
 Phone: +1.303.231.4357
 Fax: +1.303.542.5215

Terumo BCT Europe N.V.
 Europe, Middle East and Africa
 Ikaroslaan 41
 1930 Zaventem
 Belgium
 Phone: +32.2.715.0590
 Fax: +32.2.721.0770

Terumo BCT Asia Pte. Ltd.
 89 Science Park Drive
 #04-25 (Lobby B)
 The Rutherford
 Singapore 118261
 Phone: +65.6715.3778
 Fax: +65.6774.1419

Terumo BCT Latin America S.A.
 La Pampa 1517-12th Floor
 C1428DZE
 Buenos Aires
 Argentina
 Phone: +54.11.5530.5200
 Fax: +54.11.5530.5201

Terumo BCT Japan, Inc.
 Tokyo Opera City Tower 49F,
 3-20-2, Nishi-Shinjuku,
 Shinjuku-ku, Tokyo 163-1450,
 Japan
 Phone: +81.3.6743.7890
 Fax: +81.3.6743.9800

We appreciate your continued support and look forward to serving you. If you have any questions, please contact your Terumo BCT representative or your regional customer support center.

Sincerely,



Kevin Kon
Director, Regulatory Affairs

CC:

Chairman Medical Board
Heads of Department

Health Sciences Authority
11 Outram Road
169078 Singapore

14 Feb 2019

Urgent FIELD SAFETY NOTICE

IV Pole Collar for Trima Accel® Automated Blood Collection System

Name: Trima Accel® Automated Blood Collection System
Applicable Product codes: 81000

Dear Valued Customer,

Terumo BCT is dedicated to providing you with the highest quality support and remains committed to communicating information regarding Terumo BCT products. This letter provides information about the new IV pole collar accessory for the Trima Accel system.

New Trima Accel systems will be shipped with an IV pole collar (shown in Figure 1 below). This feature has been added to avoid a pinch point between the IV pole cross bar and the machine while lowering the IV pole.

If the operator does not hold onto the IV pole while pressing the IV pole release button, the IV pole may drop in a free fall. If the operator's hand or any other body part is in the path of the free-falling IV pole, the operator could be injured. Because this is an added safety feature, do not



Figure 1: Photo of the IV pole collar installed on the Trima Accel IV pole.



Figure 2: Photo of the IV pole collar prior to installation.

remove the collars after they are installed on the IV pole.

Your local Terumo BCT representatives will work with you to ensure IV pole collars are provided and installed on Trima Accel systems already in the field. On new machines, service technicians will attach the IV pole collar to the IV pole during installation.

Terumo BCT, Inc.
10811 West Collins Ave.
Lakewood, Colorado 80215-4440
USA
USA Phone: 1.877.339.4228
Phone: +1.303.231.4357
Fax: +1.303.542.5215

Terumo BCT Europe N.V.
Europe, Middle East and Africa
Ikaroslaan 41
1930 Zaventem
Belgium
Phone: +32.2.715.0590
Fax: +32.2.721.0770

Terumo BCT Asia Pte. Ltd.
89 Science Park Drive
#04-25 (Lobby B)
The Rutherford
Singapore 118261
Phone: +65.6715.3778
Fax: +65.6774.1419

Terumo BCT Latin America S.A.
La Pampa 1517-12th Floor
C1428DZE
Buenos Aires
Argentina
Phone: +54.11.5530.5200
Fax: +54.11.5530.5201

Terumo BCT Japan, Inc.
Tokyo Opera City Tower 49F,
3-20-2, Nishi-Shinjuku,
Shinjuku-ku, Tokyo 163-1450,
Japan
Phone: +81.3.6743.7890
Fax: +81.3.6743.9800

We appreciate your continued support and look forward to serving you. If you have any questions, please contact your Terumo BCT representative or your regional customer support center.

Sincerely,



Kevin Kon
Director, Regulatory Affairs

CC:

Chairman Medical Board
Heads of Department