

# Attachment 2

Supplement to the User Manual



Valid for all version of the JUPITER, ARTIS and TruSystem 7500 operating table  
TruSystem 7500 in relation to the use of a remote control JUPITER



## CAUTION

**Hazard to patient if remote control is used improperly!**

Depending on the model, using the keypad improperly may damage a key. A mechanically defective key may spontaneously triggers the corresponding operating table function on a continual basis. If damage to a key is suspected, immediately discontinue use of the remote control! If damage to a key was not noticed and the table moves unintentionally, this incorrect movement can be stopped by doing the following:

- Press any two keys simultaneously on the remote control **or**
- Press any one key on the column keypad

Hold down the selected key while simultaneously removing the defective remote control from the reception range of the operating table. Package up the defective remote control securely and send it back to TRUMPF Medizin Systeme GmbH + Co. KG in Saalfeld, Germany.

Please permanently enclose this notification to the original user manual.

V100

[www.trumpf-med.com](http://www.trumpf-med.com)

TRUMPF Medizin Systeme GmbH + Co. KG, Sitz Saalfeld, Amtsgericht Jena HRA 502248  
Unlimited Liability Partner: TRUMPF Medizin Systeme Beteiligungs-GmbH, Puchheim, Amtsgericht München HRB 139265  
Managing Board: Dr.-Ing. Eckhard Meiners

Enhancing outcomes for  
patients and their caregivers:

**Hill-Rom**